

VOCO GmbH
Knowledge Communication Department
Anton-Flettner-Str. 1-3
27472 Cuxhaven
Germany

info@voco.de
www.voco.dental



VOCO Profluorid Varnish

Product description:

VOCO Profluorid Varnish is a fluoride-containing varnish for the treatment of enamel and dentine surfaces. It is ideal for accelerated remineralisation and fluoridation of dental hard tissue.

Thanks to its natural colophony matrix, the varnish also adheres to moist surfaces and tolerates residual moisture and saliva. Furthermore, the varnish contains xylitol, which has been shown to possess a cariostatic effect, in addition to its flavour-enhancing characteristics.

Constituents (excerpt):

VOCO Profluorid Varnish contains colophony, artificial flavouring, sodium fluoride and xylitol.

1 ml of VOCO Profluorid Varnish contains 50 mg of sodium fluoride, corresponding to 22.6 mg of fluoride.

Contraindications:

- VOCO Profluorid Varnish contains colophony and artificial flavouring. It should not be used in patients with a known hypersensitivity (allergy) to these constituents.
- VOCO Profluorid Varnish should not be used in patients suffering from ulcerative gingivitis or ulcerative oral mucositis.
- VOCO Profluorid Varnish is not indicated for systemic treatment.

Patient target group:

VOCO Profluorid Varnish is suitable for use in all patients without any age or gender restrictions.

Product performance features:

The product's performance features satisfy the requirements of its intended use and the relevant product standards..

Use:

VOCO Profluorid Varnish should only be applied by a professionally trained dental practitioner.

Recommended dose:

- Treatment of patients with deciduous teeth approx. 0.25 ml of VOCO Profluorid Varnish
- Treatment of patients with mixed or permanent teeth approx. 0.40 ml of VOCO Profluorid Varnish

Apply a thin film of VOCO Profluorid Varnish to the dental hard tissue. A thick film can easily detach and offers no therapeutic advantages.

Preparation:

Clean the surface being treated thoroughly to remove all calculus. To achieve an optimal result, excess liquid and saliva should also be removed from the surface.

Application:

Apply VOCO Profluorid Varnish evenly until the complete surface is covered with a thin film. Moisten the application area (either with a gentle rinse or by utilising the natural production of saliva) in order to ensure the VOCO Profluorid Varnish sets.

Information / precautions:

- Patients must be instructed to avoid eating, drinking alcohol, cleaning their teeth and flossing for the first 4 hours after the treatment.
- Oedematous swelling has been reported in rare cases, particularly after use on larger surfaces.
- With comparable products, breathing difficulties have been reported in children with asthma in very rare cases.
- Patients who have a sensitive digestive system may experience nausea after extensive treatment.
- In the case of hypersensitivity: remove the varnish with a soft toothbrush and dental floss, then rinse with warm water.
- No other products containing fluoride should be used on the same day as the treatment.
- If the patient is taking fluoride tablets, they should stop them for a few days after treatment with VOCO Profluorid Varnish.
- Do not swallow the varnish.

Storage:

Store at 4°C to 23°C. Do not use after the expiry date.

Disposal:

Dispose of the product in accordance with local regulations.

